

Leukopettm

Avian Leukocyte Counting System

No Unopettes®? No problem!
Leukopettm is your complete
in-clinic solution for performing
WBC counts on avian species.



Each Leukopettm kit contains:

- 50 prefilled tubes containing 0.1% Phloxine.
- 25 ul Minipet for dispensing blood samples.
- 50 disposable pipet tips.

The Avian LeukopetTM test kit utilizes the same procedure, dilutions, and formulas as the original Unopette® method.

Distributed exclusively by:
Vetlab Supply 800.330.1522 • 305.232.8421 Fax • info@vetlab.com

Unopette is a trademark of Becton Dickinson, Inc.



www.vetlab.com

AVIAN LEUKOPET™ METHOD FOR PERFORMING TOTAL WHITE BLOOD CELL COUNT IN AVIAN SPECIES

Avian white blood cell morphology differs from mammalian in that the predominant polymorphonuclear leukocyte is the heterophil vs. the neutrophil in mammalian species. The remaining avian leukocytes, i.e., lymphocyte, monocyte, eosinophil, are morphologically similar to their mammalian counterparts.

Avian heterophils more closely resemble eosinophils than neutrophils. Because heterophils tend to be more abundant in the vast majority of cases, it is helpful to scan the slide prior to beginning the differential count in order to determine the specific heterophil characteristics for that particular bird. This will help to differentiate them from that bird's eosinophils. For in depth information regarding avian cell morphology, there are a number of excellent references available.

In order to determine avian total white blood cell counts, a methodology must be utilized that selectively stains heterophils and eosinophils. Avian heterophils and eosinophils both contain well-defined granules that stain orange to red. This methodology utilizes a 0.1% Phloxine solution and a standard Neubauer hemacytometer with cover glass. It is a modification of the previously published Unopette #5877 procedure which is no longer available.

Materials provided:

0.1% stabilized Phloxine solution in prefilled screw cap tubes
25 ul Minipet pipetter
Disposable pipet tips

Additional required materials*:

Hemacytometer with Neubauer ruling
Hemacytometer cover glass
Microscope
Single or multi-place tabulator

*Available from Vetlab® Supply

Storage requirements

Keep in dark, cool area. Exposure to light can cause breakdown of the Phloxine stain.

PROCEDURE

1. Make a blood smear by routine methods and stain with Wright Giemsa or another appropriate differential stain.
2. Perform the differential blood count as for mammalian species substituting the heterophil for the neutrophil. Record results.
3. Attach a clean, unused disposable pipet tip to the 25 ul pipetter.
4. Unscrew the cap of one of the prefilled Phloxine tubes and place in a tube rack.
5. Using the pipetter, aspirate 25 ul of freshly drawn anticoagulated blood.
6. Dispense the blood sample into the tube of phloxine and rinse the pipet tip **thoroughly** by aspirating and dispensing the phloxine/blood solution at least 6 times. **IMPORTANT:** It is critical that all of the blood be rinsed from the pipet tip to insure a proper dilution. Depending on the viscosity of the sample, this may require additional rinsing of the tip.
7. Cap the tube and mix well by inverting several times. Do not shake.
8. Allow tube to incubate for 10 minutes but no longer than 1 hour.
9. Using the rinsed pipet, aspirate a sample from the tube and charge the hemacytometer.
10. Allow to stand for up to 10 minutes for cells to settle.
11. Using the 10X objective, count the heterophils and eosinophils in both chambers of the Neubauer hemacytometer.
12. Use the following formula to calculate the total leukocyte count:

$$\text{Total WBC/ul} = \frac{\text{Total Heterophil + Eosinophil (both chambers)} \times 1.1 \times 16 \times 100}{\% \text{Heterophils} + \% \text{Eosinophils (from differential count)}}$$

References:

Campbell, T.W. Avian Hematology and Cytology, 2nd edition. Iowa State University Press, Ames, IA, 1995, pp 3-5

For reorders and support contact:

Vetlab Supply
18131 SW 98th Ct.
Palmetto Bay, FL 33157
800.330.1522 • 305.232.8421 Fax • info@vetlab.com
www.vetlab.com

Legal Notice: Avian Leukopet® has not been validated in any controlled animal studies. Although side by side comparisons with the original Becton Dickinson Eosinophil Unopette® #5877 have yielded comparable results, the manufacturer makes no claims of performance or suitability for its purpose. The responsibility for achieving accurate and clinically useful results with Avian Leukopet® rests solely with the Veterinarian and/or individual performing the test. Avian Leukopet® has not been approved for use in human testing.