

# LipoClear<sup>®</sup>

## 95 Second lipemic sample clearing reagent

### Safe-Effective Process


- Inexpensive and convenient
- Non toxic, non-carcinogenic reagent, assures safe handling
- Most common analytes are not affected including enzymes and bilirubin

### Available in convenient pre-filled tubes

- Low cost reagent is pre-dispensed into microtubes, no measuring required
- Two sample sizes available



The StatSpin high speed centrifuge is a perfect system for clearing lipemic samples in 95 seconds.

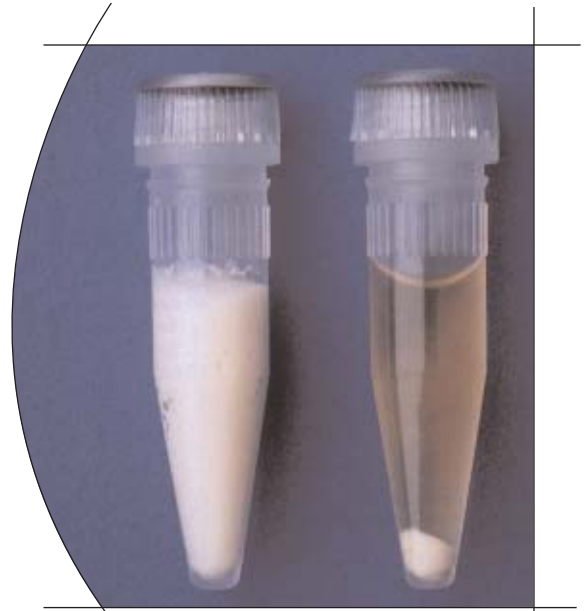

LipoClear reagent clears quickly and conveniently.

### Simple Procedure

- Add lipemic sample according to recommended procedure to reagent pre-filled tube
- Mix well and let stand for 5 minutes
- Centrifuge and remove clarified sample for analysis
- Multiply results by 1.2